

KOWA SL-19 plus

KOWA SL-19 App



 Cybersecurity

The icon is a black shield with a white checkmark inside, representing cybersecurity.

The KOWA SL-19 App is an application software used for the capture, display and recording of images using the camera version of the KOWA SL-19 hand-held slit lamp (SL-19 plus). Please purchase the SL-19 plus prior to using this application. For information on how to use the SL-19 plus and associated precautions, kindly refer to the SL-19 plus instruction manual.

This document describes recommendations for improving cybersecurity when using this application.

Please note that these instructions are written under the assumption that you are familiar with the iPad mobile device on which the application is installed.

This application software is not classed as a Medical Device. Displayed/Recorded imaging data (including enlarged images) are for reference use only. Diagnosis should only be made by direct visual inspection through the SL-19 plus.

Operation of this application does not normally cause damage to the SL-19 plus, however if you experience any abnormality during use, kindly contact us or your Kowa distributor immediately.

Operating environment of this application

This application is for the iPad series running iPadOS operating system software version 15.4 or later.

Disclaimer

Kowa Company shall not be liable for accidents caused by the iPad Mobile Device on which this application is installed.

Kowa Company can not be held responsible for any damage such as data corruption or loss of data on the iPad Mobile Device due to the use of this application.

When using the data sharing function, please ensure that the data being shared is sent to the correct destination. Kowa Company can not be held responsible for any damage caused when sharing data.

This document is based on information accurate as of the copyright date.

Recommended operating environments, functions, etc. are subject to change without notice. Please note that the contents of this document may differ.

The application operates using a one-to-one wireless LAN connection with the SL-19 plus. Recorded images may be transferred to other personal computers, devices etc... When connecting to a network using the sharing function, kindly ensure that cybersecurity is present.

iPad settings

We recommend applying the following settings for your iPad:

- Set a "passcode" to unlock the device.
- Set "Auto Lock" to something other than "None" and longer than the anticipated recording time, such as 15 minutes. Please note that if the auto lock time is too short, your device may lock and recording will be terminated.
- Turn on the "Find My iPad". If the iPad is lost or stolen, you can mark it as lost in the "Find My App".

These settings prevent third parties from using the iPad and from accessing all data stored on the iPad in the event that it is lost or stolen. For setup instructions and how to locate the iPad if it is lost or stolen, please visit the official "Apple Support" website. When exporting over a wireless LAN, it is recommended to connect to an encrypted wireless LAN.

Precautions for use

When leaving the iPad unattended, it is recommended that the device is locked.

When sharing data using the AirDrop software feature, email, etc., be careful not to send it to the wrong destination.

When swapping iPads

Backup and restore allows you to transfer all the content, settings, and Apps on the current iPad to the new iPad.

To back up and restore, follow the steps "Back up your iPad" and "Restore all content from a backup to your iPad" in the iPad User Guide. For instructions, refer to the official "Apple Support" website.

When creating a backup using a Mac or Windows® PC, it is recommended that data is encrypted and a back-up is saved.

When discarding the iPad or no longer using this App

When discarding the iPad or no longer using the App, be sure to remove ALL content, settings, and Apps stored on the iPad to ensure all data, including personal information is erased.

Simply uninstalling the SL-19 App will not completely erase the data stored within it. Data may be recovered and used by malicious users.

The "Erase your iPad" procedure in the iPad User Guide permanently erases all content, settings, and apps, including app data, on the iPad.

Before erasing your iPad, please read the disclaimer on page 2 and save any necessary content, settings, and Apps on your iPad.

To save the data of this App, please use the "Export App Function." To save content, settings, and Apps on the iPad, follow the steps in the "Back up your iPad" section of the iPad User Guide.

For further instructions, please refer to the official "Apple Support" website.



Technology for Life Science



World Sales Headquarters

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo 103-8433, Japan

Phone : +81(3)3279-7639

Facsimile : +81(3)3279-7673

E-mail : life@kowa.co.jp

URL : <https://www.kowa.co.jp/e-life/>

Microsoft® and Windows® are registered trademark or trademark of
Microsoft Corporation in the United States and other countries.

Mac and macOS are trademarks of Apple Inc.

© 2024 Kowa Company, Ltd. All rights reserved.
SL-AP192 S1 V1.1E 241129 KW
Printed in Japan